




OLIDENT
INNOVATION

OliEtch BOND

ONE LAYER - ZERO MISTAKES

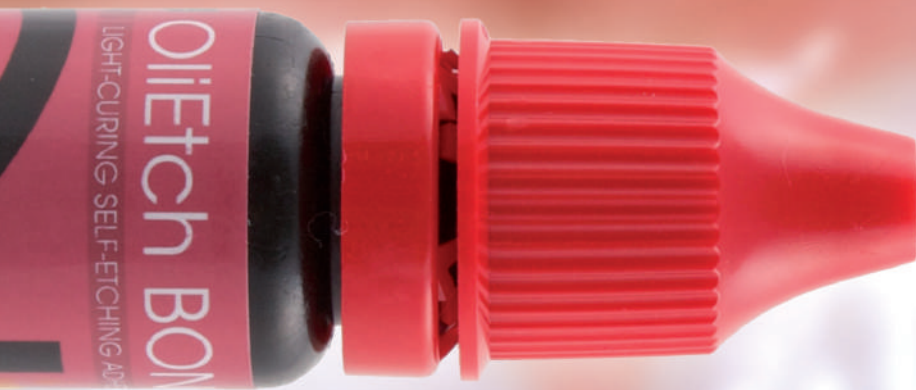

- one-component
- single layer
- self-etching
- light-curing



OliEtch BOND

ONE layer - ZERO mistakes:

OliEtch Bond is a one component, single layer 7th generation bonding system. It was created to simplify the procedure, while maintaining excellent quality and assurance of bonding strength. The simple technique eliminates any procedural errors.




code: **OL003** - bottle 5 ml

ONE procedure in 50 seconds!


Earlier generations of bonding systems required the use of several components / bottles and several separate procedures. With **OliEtch Bond** all the procedures: etching, conditioning and bonding take place in one stage and takes only 50 seconds!




Apply a small amount of **OliEtch BOND** on the tray.

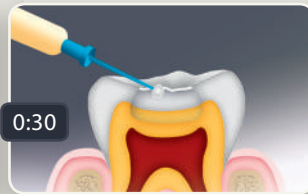

Use the applicator to thoroughly rub **OliEtch BOND** in moist enamel and dentin for 30 seconds

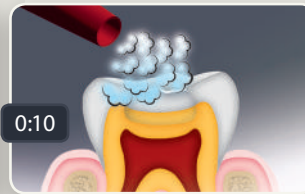

Blow air for 10 seconds to remove any excess **OliEtch BOND**.

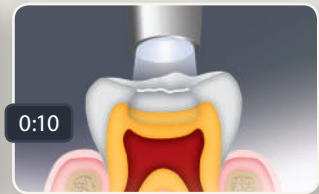


Polimerise **OliEtch BOND** with a curing-light for 10 seconds.


ONE of the strongest - check and compare

Among the most popular self-adhesive systems, OliEtch BOND stands out with its excellent bond strength value. Only one layer of OliEtch Bond is enough to get a strong and long-lasting bond of light-curing composites to enamel and dentin.*

The bonding strength to enamel (MPa)*

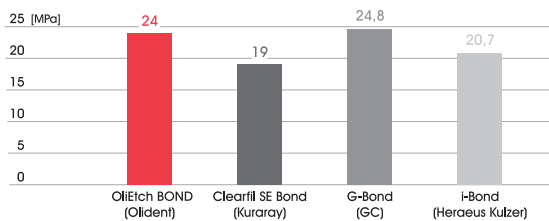

The bonding strength to dentin (MPa)*

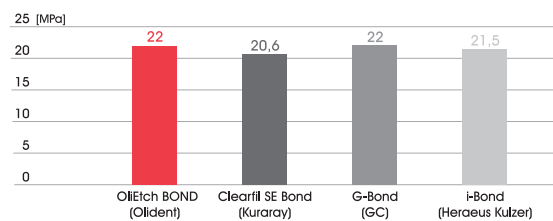

* Internal research conducted by the manufacturer, Germany, 2011.


SPECIAL Price for German quality

The **German Quality** mark is a guarantee of reliable quality of each OliEtch Bond bottle. Thanks to advanced technology we are also able to offer the product for an exceptionally affordable price!

Ask our Representative for details.



Your business Partner

 Join us on Facebook

OLIDENT sp. z o. o., Sp. k.
 Podleze 653
 32-003 Podleze
 tel. +48 12 29 00 502
 fax: +48 12 29 00 325
 info@olident.com
 www.olident.com



OLIDENT



EN ISO
13485:2012
+AC:2012

