

## PRODUCT INFORMATION

**OliNano Seal** is a silicone based one component acid protecting varnish with Nano-Fluorapatite and Nano-Calciumfluoride for sealing of teeth. Nano-technology fluorapatite particles are known to enhance remineralization of tooth material.

**OliNano Seal** does not require etching. It builds a protective film, that can be renewed or repaired just as you like, if necessary. The application is easy and time saving.

## INDICATIONS

- sealing of teeth where cleaning is difficult due to multi-band technique
- sealing of teeth when patients are handicapped
- temporary sealing of restorations with marginal leakage

## APPLICATION

The teeth are cleaned thoroughly with an oil-free paste, polished and dried in an oil- and water free air stream.

**Before application vigorous shake the bottle with the OliNano Seal. During shaking listen to the glass bead inside the bottle.**

Apply OliNano Seal 2 – 3 times in an even thin layer. Avoid contact of the OliNano Seal with the mucosa.

Let OliNano Seal dry for 1 minute or dry it gentle in an oil-free air stream.

Close the bottle immediately after use.

## IMPORTANT REMARKS

- In the first 2 hours after application of OliNano Seal don't eat anything nor brush the teeth.
- In case of protecting teeth with OliNano Seal during application of orthodontic multi-band technique, the brackets are placed on the teeth **before** the application of OliNano Seal.
- The intense smell of OliNano Seal volatizes during drying within 1 minute.
- OliNano Seal contains ethylacetate. Ethylacetate is highly flammable. Keep away from sources of ignition. Avoid inhalation of ethylacetate vapour. After eye contact rinse out with plenty of water und call an ophthalmologist. After contact with skin, wash with plenty of water and soap.
- Avoid contamination of the OliNano Seal with dirty disposable brushes.
- Impressions with vinyl silicones can be made earliest 4 weeks after the treatment with OliNano Seal.
- Due to the ethyl acetate content, it is recommended to use only the original accessories included with the product.

## STORAGE

Do not store above 25 °C (77 °F)! Do not use after expiration date!

Store OliNano Seal tightly closed in a well ventilated place.

Keep away from children!

## ADDITIONAL INFORMATION

For dental use only!

## PACKAGING UND REFERENCE NUMBER

1 bottle 5 ml, accessories, this instruction for use	<b>OL008</b>
--	--------------



Version 05/2019 (2019-10-29)

Reservation:

Olident Co. Ltd., L. P. warrants this product will be free from defects in material and manufacture. Olident Co. Ltd., L. P. makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and Olident Co. Ltd., L. P.'s sole obligation shall be repair or replacement of the Olident Co. Ltd., L. P. product.

Limitation of liability - except where prohibited by law, Olident Co. Ltd., L. P. will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability. The CE 0197 mark does not apply to accessories

 OLIDENT Sp. z o. o., Sp. k.  
32-003 Podtęże  
Podtęże 653  
tel.: +48/12/650 70 10  
info@olident.com  
www.olident.com  
www.olidentdentalshop.pl



© OLIDENT® 2019



Made in EU

